


DENTAL SUPPORT RING

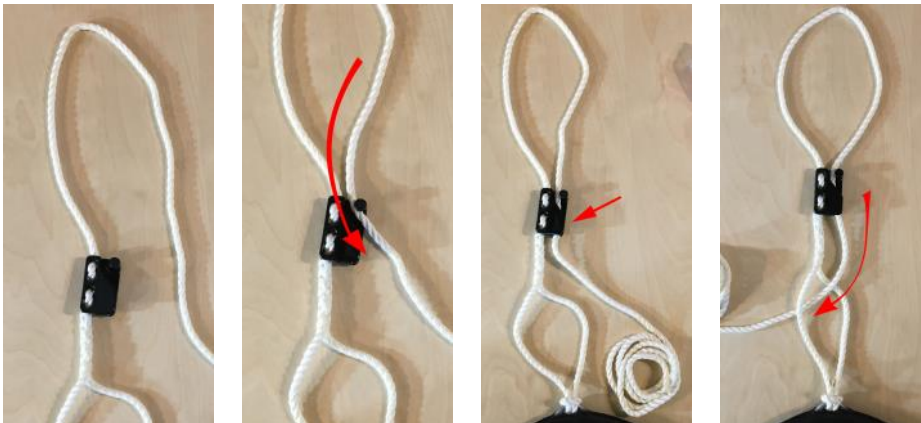

Securing the rope

1. make sure the overhead hook or beam is strong enough to support the horse's head. The black cleat can be moved as needed.
2. Feed the rope through the upper point of the cleat
3. Secure the rope in the cleat
4. By feeding the rope through the loop you minimize the risk of accidental release

This support ring helps positioning the horse's head for equine dentistry.

Please keep the ring clean and treat the leather cover as needed.

Make sure you have the horse restrained in stocks and well sedated before using this dental support ring.

Make sure the full mouth speculum is designed to work with this support ring, especially make sure the speculum does not accidentally close when resting on the pivoting base.

Contact Swissvet for questions.